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
Original operating instructions

First steps

What do these operating instructions apply to?

These operating instructions apply to the following devices, which differ in their range of functions or equipment features: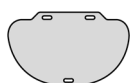

FN 6864/15 Pressure mat for room radio
 FN 6064/15 Pressure mat for room bus

Identifying the device

Make sure that these operating instructions belong to your device. These operating instructions do not apply to other devices, even if they look similar or appear to be of the same design.

Unique feature: Type plate ("x" = any number)

Position of the type plate:


On the underside of the pressure mat

Checking the scope of delivery

Check that the scope of delivery is complete and in perfect condition. In case of missing parts or damage: Do not connect the device and do not put it into operation! Make a complaint to the supplier.

The scope of delivery consists of: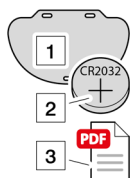

- 1 Pressure mat
- 2 Battery CR2032 (built-in)
- 3 Download the latest version of the operating instructions at <https://www.tetronik-kommunikationstechnik.de/downloads/original-betriebsanleitungen/>. Search term: FN number of your device.

What to do with the packaging?

If the device or parts of it are to be stored before or after use: Keep the packaging.

If nothing is to be stored:

Dispose of the packaging in accordance with regional regulations. Do not dispose of the packaging in household waste!

Using the operating instructions

Read first!

Read these operating instructions in full. Keep the operating instructions in a safe place. If you have any questions, please contact the manufacturer.

Interpreting signal words

Pay attention to signal words that warn of dangers. Follow the instructions for risk avoidance.

Signal word	Meaning
 DANGER	Warns of imminent danger to life or danger of serious injury.
 WARNING	Warns of possible imminent danger to life or danger of serious injury.
 CAUTION	Warns of possible imminent danger of minor or slight injuries.
 SAFETY INSTRUCTIONS	Warns of possible material damage.
 NOTICE	Indicates additional information.

Information about the device


Intended use

The pressure mat is used to protect those who are disorientated. When stepped on, it sends a message to the associated receiver and triggers a call in the call system. Application examples:

- Monitoring getting up from bed
- Monitoring passing through a door

Any other use is contrary to the intended use and may lead to malfunctions and damage to this and connected devices. Disregarding these operating instructions also constitutes improper use.

Arrangement in the call system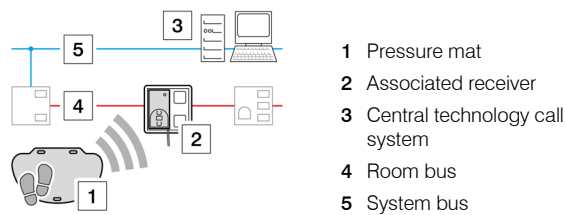


Features and properties

- Call triggering by walking or driving over the mat
- Sturdy, non-slip and non-trip design
- Surface with non-slip studded structure
- Waterproof design, also suitable for wet rooms
- Low installation height
- Sloped edges for barrier-free access with rollator or wheelchair
- Reliable function even when used under a light carpet
- Fail-safe thanks to early "low battery" signal

Controls and displays

The device has the following controls and displays:

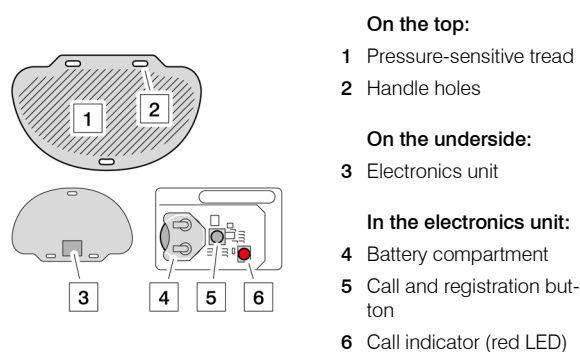Technical data

Physical data

Dimensions (W x H x D)	1,100 x 9 x 700 mm
Height around electronics unit	15 mm
Weight	approx. 7,500 g
Trigger weight	approx. 10 kg
Colour	Care yellow

Electrical data

Supply voltage	Battery type CR2032; 3.0 V
Battery life	approx. 3 years
Operating frequency	433.92 MHz
Protection class	IP 54

Ambient conditions

Permissible ambient temperature	0... +40°C
Permissible storage temperature	-10... +60°C
Relative humidity	10...85% (without condensation)

The device is suitable for use in damp conditions, e.g. outdoors and in wet rooms.

The device is only suitable for use in shielded rooms (e.g. X-ray rooms) if the associated receiver is located in the same room.

The device must not be used in potentially explosive atmospheres.


The declaration of conformity can be requested from the manufacturer. See last page for contact details.

Normative information

- Planning and testing of the call system and certification of its functionality must be carried out by a "specialist planner for call systems".
- This device is not a medical device within the meaning of Directive 93/42/EEC.
- The electrical safety of the call system is guaranteed by system separation.

Instructions for technical staff


Procuring the required accessories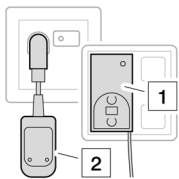

- 1 FN 6864/00
IMA receiver for room radio
- 2 Receiver for wireless handheld button
FN 6064/10 7-pole DIN
FN 6064/50 8-pole mini-DIN

Activating the battery

The pressure mat is supplied with an inserted battery separator film to secure it during transport. The battery separator film must be removed for commissioning.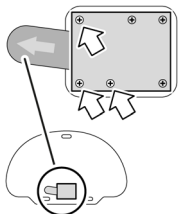


- 1 Loosen three screws on the cover.
 - 2 Pull out the battery separator film.
 - 3 Tighten the three screws again hand-tight.
- The device is ready for commissioning.


Putting the device into operation

Ensure that the following requirements are met:

- The operating instructions for the corresponding receiver are to hand.
- The associated receiver operates on the same frequency as the pressure mat.
- The associated receiver is ready for use and is within reach of the pressure mat.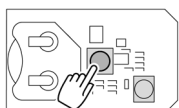

- 1 To trigger a call: Step on the pressure mat.
Or press the registration button when the electronics unit is open.

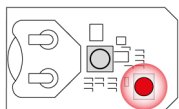
- The device sends a call.
Feature: The red LED flashes 5 times (call indicator).
- 2 Continue the registration process as described in the operating instructions for the receiver.
The pressure mat is then ready for use.

Replacing the battery


Battery type: CR2032


The battery compartment is located on the underside of the pressure mat.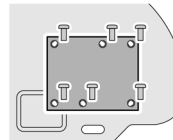

- 1 Remove the cover of the electronics unit.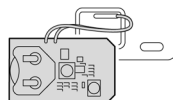


- 2 Remove the circuit board.

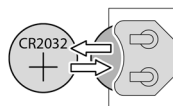
- 3 Replace the battery.
Ensure correct polarity (positive pole at the top).

- 4 Reinsert the circuit board.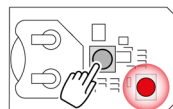

- 5 To check the function: Trigger a call.
The red LED must light up 5 times (call indicator).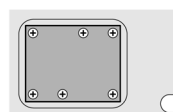

- 6 Close the cover of the electronics unit. Caution: Do not trap the cable!
Tighten the screws hand-tight.

De-registering the device

To de-register the device, see the operating instructions for the associated receiver.

Servicing and maintaining the device

Ensure that this device and the entire call system are fail-safe at all times. Follow the regulation DIN VDE 0834-1:2016-06 (11.2 to 11.4).

Repairing the device

SAFETY INSTRUCTIONS


The device fulfils safety-relevant tasks.

There is a risk of malfunction if the device is repaired incorrectly.

Only have repairs carried out by the manufacturer.

Instructions for operating staff


Positioning the pressure mat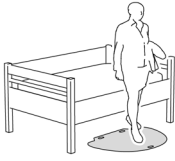

Place the pressure mat where the resident or patient should trigger a call by stepping on it.

Studded structure upwards!

Example: Next to the bed at night.

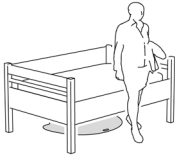If call triggering is not necessary, place the pressure mat where it cannot be walked on.

Example: Under the bed during the day.

Instructions for auxiliary staff

Cleaning

Clean the surfaces with a damp, lint-free cloth. Do not use any aggressive or abrasive cleaning agents.

Disinfecting

Disinfect the surface of the device with a commercially available surface disinfectant according to the instructions.

SAFETY INSTRUCTIONS

The device contains heat-sensitive components.
There is a risk of destruction when exposed to heat.
Do not sterilise the device thermally.

Procuring consumables

SAFETY INSTRUCTIONS

The device fulfils safety-relevant tasks.
There is a risk of malfunction if foreign material is used.
Only use original material from the manufacturer.

The following consumables are available:

Battery type CR2032; 3.0 V

Storing the pressure mat when not in use

SAFETY INSTRUCTIONS

The pressure mat has a pressure-sensitive mesh on the inside. If stored incorrectly, the function is impaired or lost completely.

- Always store the pressure mat lying flat.
- Do not roll or bend the pressure mat.
- Do not let the pressure mat hang over an edge.
- Do not place any other objects on the pressure mat.

For the requirements regarding the ambient conditions of the storage location, see "Technical data".

Disposing of the device

At the end of the service life:

Observe the regional disposal regulations. Have the device recycled properly. Do not dispose of the device with household waste!